

CM TraumaMattress™

CM ProLine



- ✓ SAFE AND STABLE
IMMOBILISATION
- ✓ X-STRAPS
- ✓ SAFE AND EASY
TRANSFER
- ✓ REMOVABLE
COVERING
- ✓ PRESSURE-RELIEVING
COMFORTABLE

CM TraumaMattress™ is a pressure-relieving intrahospital traumamattress designed for fast and safe immobilisation and transferring of trauma patients.



Ergonomic handles and lowfriction underside makes transfer to CT and X-ray table easy.

PRODUCT INFORMATION:

Intrahospital trauma mattress, designed for immobilisation and transfer of patients. CT-compatible. Low attenuating of x-ray. The waterproof core of the mattress consists of a carbonfibre sandwich and a 40 mm pressure-relieving layer of polyether and viscoelastic foam. Removeable covering consisting of durable polyester-weave and a underside of silicone-coated polyamide with low friction. Ergonomic handles made of glassfibre. CM TraumaMattress does not contain any metal.

ADULT MODEL:

Size: : 200 x 53 x 6 cm Weight: ca 8 kg
Art.nr: CMTRM

Name: CM TraumaMattress with straps

Accessories adult:

Covering for adult model	Art.nr: CMTR1
Stabilisations cushions, set	Art.nr: CMTR2
Body straps, complete set	Art.nr: CMTR3/4
CM TraumaCover	Art.nr: CMCOVER
CM T-Fix Head Fixation	Art.nr: CMTFIX
CM ProSlide Slide Board	Art.nr: CMPRO

CHILDREN MODEL:

Size: 142 x 44 x 6 cm Weight: ca 5 kg
Art.nr: CMTRMM

Name: CM TraumaMattress Mini with straps

Accessories children:

Covering for child model	Art.nr: CMTR1M
Stabilisation cushions, set	Art.nr: CMTR2
Body straps, complete set	Art.nr: CMTR3M
CM TraumaCover Mini	Art.nr: CMCOVERM



Made in Sweden



CM TraumaCover™ Single use cover, with head fixation
Art.nr:
CMCOVER (adult, 15-pack)
CMCOVERM (child, 10-pack)



CM T-Fix™ Head Fixation
Art.nr: CMTFIX (25-pack)
Only adult

COMFORT medical

FAQ

What are the benefits of the CM TraumaMattress?

- 1) The CM TraumaMattress eliminates difficult and potentially dangerous transfer and movement of injured patients. Saving time on patient transfer increases patient safety and speeds treatment.
- 2) The special padding reduces the risk of pressure injuries for patients that may be on the CM TraumaMattres for long periods of time.
- 3) If the original TraumaMattress covering is damage it can be replaced , order number CMTR1.
- 4) X-straps gives good immobilisation of the upper body.

How to use the CM Trauma Mattress:

The CM TraumaMattress use starts in the hospital emergency department upon admission of an injured patient.

Moving the patient onto the CM TraumaMattress:

- 1) If the patient is transferred onto the CM TraumaMattress from a vacuum mattress, using a product such as the CM ProSlide or rollboard will assist.
- 2) If the patient is on a spineboard or scoop stretcher, you can place these directly on top of the CM TraumaMattress but remove the patient from these as soon as possible to realize the full benefit of the pressure relieving capabilities of the CM TraumaMattress.

How to apply the CM TramaTransfer Cover:

- 1) Pull the cover over the head part of the CM TraumaMattress and stretch it downward over the mattress.
- 2) Tighten the Velcro straps firmly around the mattress and secure them to the cover.
- 3) The CMCOVER is a single-use product made of polyurethane and polyester. Each CMCOVER comes with a single-use T-Fix head immobilizer.

How to secure the patient on the CM TraumaMattress:

- 1) Place the patient centrally on the CM TraumaMattress with the patient's head located to allow for head fixation using the T-Fix part of the CM TraumaMattress Cover.
- 2) Use additional cushions, if extra stabilization is needed along the sides of the body.
- 3) Place the head cushion (T-Fix) securely around the patient's head and firmly press the bottom Velcro onto the cover.
- 4) Secure the patient's head using the white Velcro straps and attach the white straps to the sides of the cushion and to the red Velcro straps.
- 5) Wrap the blanket part of the CM cover around the patient as treatment procedures allow.

How to clean the CM TraumaMattress:

- 1) The body straps can be washed from blood in cold water and after that washed in machine. Disinfection use following hospital policy. The body straps can be replaced as needed, order number CMTR3 (4 straps).
- 2) The mattress covering and other parts can be wiped clean with disinfectant per product instructions for use and following hospital policy.
- 3) If necessary, the mattress covering can be removed for more thorough cleaning with disinfectants. Replacement mattress covering is available, order number CMTR1.

ACCESSORIES



Mattress covering, with handles
Art.nr: CMTR1 (adult)
CMTR1M (child)



Stabilisation Cushions (set with 4 pcs)
Art.nr: CMTR2



Straps for body, complete set
Art.nr: CMTR3/4 (adult)
CMTR3M (child)



ProSlide Slide Board
Art.nr: CMPRO

MANUFACTURER:
Comfort Medical AB

Mail: info@comfortmedical.se
www.comfortmedical.se

COMFORT medical

DISTRIBUTOR: